



# Dr. Prepare

## 👉 THE MANUAL OF USB HEATING EYE MASK

- Product materials: jacket with leather and cotton. Inner lining with mesh fabric
- Heating temperature: 95°F-122°F / 35°C-50°C(room temperature 68°F / 20°C)
- Product size: 7.68\*4.1 in
- Product weight: 2.7 / 3.5 oz
- Rated power: 5W
- Rated voltage: 5V(USB)1A

## 👉 PRODUCT FEATURES

- Soothing effect of heat: promote blood circulation around the eyes, effectively relieve the dark circles,bags under the eyes and fine lines, improve dry eyes, pain and other eye problems.
- Product materials: Jacket with leather and cotton, inner lining with mesh fabric
- Removable and washable design: Zippered design, easy to take out the lining to clean the jacket.(thelining can not be washed)

## 👉 TIP



To enhance the steam effect spray warm water on the eye patch on the eye mask and warm it for five minutes before use.







Choose suitable temperature, set a timer, and you stait using you eye mask, Recommended temperature and time : 113-122°F / 45-50°C. for 20 minutes.



After heating,itis recomended to apply a cold compress for 5 minutes to minimize pores and to prevent skin problems.

## OPERATION MANUAL

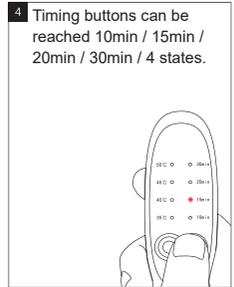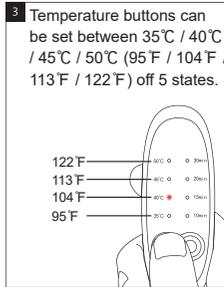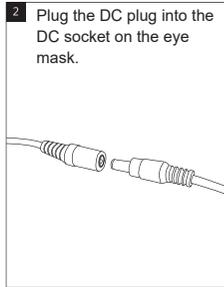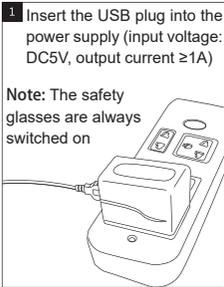

## PRECAUTIONS

- This product can be connected to the computer or your phone charger. If the output of your power supply does not meet product's rated input (5V, 1A), your eye mask may not work properly.
- This product has been tested to be safe and reliable when it is heating. You can feel free to choose suitable temperature for a soothing spa for eyes.
- Before you go to sleep, we advise you to set a timer from 15-30 minutes to protect your eyes from scald.
- This eye mask is washable. Open the zipper on the jacket and take out the heating pad before washing. Handwash the mask in water under 40°C / 104°F. Dry cleaning and ironing is not recommended.
- If you have eye diseases, please use this product under your doctor's instructions.
- Do not use your eye mask when it is soaked through.
- If you feel unwell (skin allergies, etc.) after using this product, please stop using the eye mask.
- Do not fold the eye mask when it is operating or is still lukewarm after shutting down.
- High heating temperature for long-time use is not recommended because it may cause discomfort of your skin and shorten the lifespan of your eye mask.
- Do not put extra things on your eye mask while it is operating.
- Keep this product away from children. For people who are unable to take care of themselves or not susceptible to heat, we advise them not to use this product unless their guardians have preset the mask or given them full instructions.