# Dr. Prepare

<b>EN</b>	<b>HEATED MASSAGE EYE MASK</b> .....	<b>01</b>
	<b>USER GUIDE</b>	
<b>DE</b>	<b>DIE BEHEIZTE MASSAGE-SCHLAFMASKE</b> .....	<b>06</b>
	<b>DIE BEDIENUNGSANLEITUNG</b>	
<b>JP</b>	<b>ホットアイマッサージャー</b> .....	<b>11</b>
	<b>ユーザーガイド</b>	

<b>UK</b>	<b>REP</b>
-----------	------------

Name: Sea&Mew Accounting Ltd  
Address: Electric Avenue Vision 25, London,  
Enfield EN3 7GD  
Contact Person: Billy Han  
Tel: +44 7399648608  
E-mail: info@seamew.net

<b>EC</b>	<b>REP</b>
-----------	------------

Name: Sea&Mew Consulting GmbH  
Address: Mittenhuber Straße 4, 92318 Neumarkt  
Contact Person: Yuan Li  
Tel: +49 15224685061  
E-mail: Compliance.EU@outlook.com

Manufacturer / Importer: Dr Prepare Global Limited  
Address: HK, 999077, HONGKONG, CANTON RD TST KLN, RM 803 LIPPO SUN PLAZA 28  
Contact Person: Michelle Guo  
Tel: +86 152 6258 8612

# **HEATED MASSAGE EYE MASK**

Model: DMEM01

## SPECIFICATIONS

Product Material	Polyamide Fiber, Spandex
Heating Element Material	Graphene
Rated Power	6W
Input	Max. 5V/2A (USB)
Heating Level	01 (113°F / 45°C) 02 (131°F / 55°C) 03 (149°F / 65°C)
Timer	30 / 45 / 60 minutes
Temperature Controller Cord	4.92 ft / 1.5m
Product Size	10.92*3.71*0.89 Inch / 27.74*9.42*2.26 cm

## PACKAGE INCLUDES

- 1 x Heated Eye Mask
- 1 x Temperature Controller
- 1 x Storage Bag
- 1 x Spray Bottle
- 2 x Earplug
- 1 x User Manual

## INSTRUCTIONS FOR TEMPERATURE CONTROLLER

1. Take out the temperature controller, and attach magnetic connector to the magnetic plate on the eye mask.

2. Insert the USB plug into the power supply (power bank, power adapter, laptop, etc.). After power on, the display on the temperature controller will light up for about 2 seconds and then turns off. At the same time, the massage button on the eye mask will cycle through red, blue and green light and then goes out.

3. Press the heating button to turn on the eye mask and the heating function. When starting, it defaults to heat on the highest level and the display shows "03". Press the heating button to switch from Level 03, 02, 01. Press and hold the heating button for 2 seconds to turn off the heating function.

4. Press the timer button to set the timer (30 / 45 / 60 minutes). The default timer is 30 minutes, and the temperature controller display shows "30"; press again to set the timer to 45 minutes, the display will show "45"; press again to set the timer to 60 minutes, the display will show "60". The display will automatically show the remaining time without further operation for 10 seconds. Press the button again to turn off the timer, the display will show "00" and keep heating.

## INSTRUCTIONS FOR MESSAGE BUTTON

The heating function and massage function works independently. The massage function can be turned on separately, or it can be turned on when the heating function is on. The massage function will not stop working when the timer expires.

- When power on, press and hold the message button for 2 seconds to turn on / off the massage function.

- The default massage mode is normal mode (with blue light on). Press the massage button to switch from normal mode (with red light on), leisure mode (with blue light on), vitality mode (with green light on).

## PRECAUTIONS

- Washing is not recommended. Wipe and clean the surface of the eye mask with wet wipes.
- It is not recommended to fold the eye mask as it may damage the heating element inside.

## WARRANTY

- Dr. Prepare products are covered by a 12-month limited warranty from the original purchase date. If any problems occur, please contact our support team.
- We only provide after-sales services for products that are sold by Dr. Prepare or retailers and distributors authorized by Dr. Prepare. If you have purchased your unit from other channels, please contact your seller for more information about return and warranty.

## CUSTOMER SUPPORT

- If you have any questions or concerns, please send us an email at [support@drprepare.com](mailto:support@drprepare.com)

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